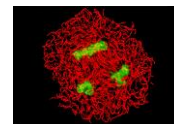
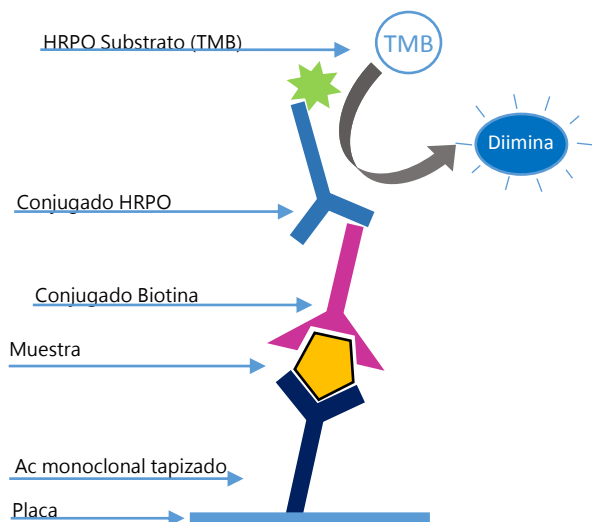
INgezim BVD DAS

R.12.BVD.K2

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
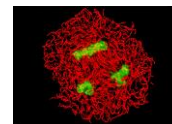
INgezim BVD DAS

R.12.BVD.K2

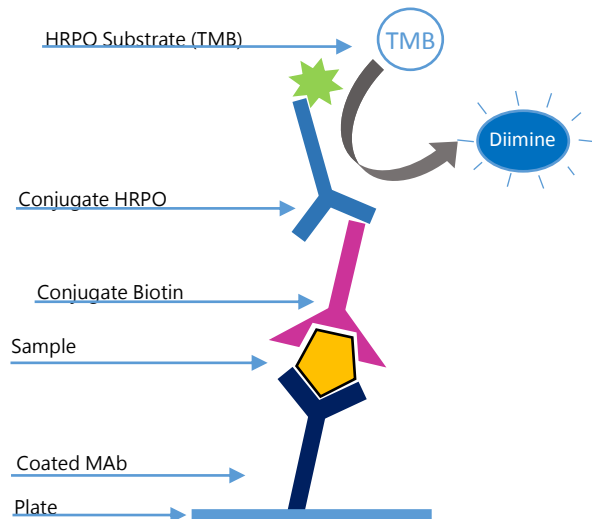INgezim BVD DAS is an enzymatic assay based on a Double Antibody Sandwich ELISA technique, which uses monoclonal antibodies (MAb) specific of Bovine Viral Diarrhoea Virus (BVDV) p80/p125 protein.

TECHNICAL BASIS OF THE KIT

1. Plates are coated with a monoclonal antibody specific of BVDV protein p80/p125. Samples are added and incubated.
2. If the samples contain BVDV, they will bind to the specific monoclonal antibody.
3. After an incubation period, a mix of MAb-Biotin specific to BVDV's p80/p125 is added. The mix will bind to the protein if there is antigen in the sample.
4. To detect this binding, Streptavidine-HRPO is added.
5. The binding between Streptavidine-Biotin is detected by a colorimetric reaction after the addition of a substrate specific of HRPO.

APPLICATION

- Detection of BVDV antigen p80/p125 in blood, sera, plasma, leukocytes, ear tissue and cell culture.
- The combined use of this kit with another one for antibodies detection allows detecting IPIs.

INTERPRETATION OF THE RESULTS

One cut off is used for the results interpretation. Samples are **Positive** if their OD value is higher than the cut off. Samples are **Negative** if their OD value is lower than the cut off.

VALIDATION

Correlation with other commercial kits

1. A set of 341 samples from cattle, classified as positive/negative by a commercial assay (product 1) was analyzed. The results obtained indicated 98.8% correspondence between both assays.
2. A set of 53 samples from cattle, classified as positive/negative by a commercial assay (product 2) was analyzed. The results obtained indicated 98.1% correspondence between both assays.
3. A set of 310 samples of ear tissue of cattle, classified as positive/negative by other assays was analyzed. The results obtained indicated 90% correspondence between assays.

		INgezim BVD DAS		
		POS	NEG	DUD
PRODUCT 1	POS	7	0	0
	NEG	0	330	0
	DUD	1	3	0

		INgezim BVD DAS		
		POS	NEG	DUD
PRODUCT 2	POS	12	1	0
	NEG	0	40	0
	DUD	0	0	0

		INgezim BVD DAS		
		POS	NEG	TOT
OTHER TECHNIQUES	POS	120	30	150
	NEG	0	160	160
	TOT	120	190	310

COMPOSITION OF THE KIT

- Microtitration plate of 96 wells
- Vial with Positive Control
- Vial with Negative Control
- Vial with Biotin Conjugate
- Vial with Streptavidine-HRPO
- Bottle with Washing Solution
- Bottle with diluent
- Bottle with substrate (TMB)
- Bottle with stop solution
- Bottle with Extraction Buffer

REGISTRATION NUMBER 2662 RD
PRODUCT MANUFACTURED BY INGENASA

SHELF LIFE: **24 months**
Stored at 2°C-8°C

Ed.030619